RELIANCE ORTHODONTIC PRODUCTS, INC.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only

RelianceOrthodontics.com

RelianceOrthodontics.com

ITC TRANSLATIONS

ITC USA

2000 PGA Blvd., Suite 4440

Palm Beach Gardens, FL 33408

⚠️ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

- ⚠️ Corrosive

⚠️ Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices. Protective eyewear should be worn by the patient and dental staff.

⚠️ **TEMPERATURE LIMITATION:** Store at room temperature (15-25°C).

PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

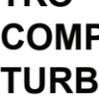
CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RELIANCE

TKO™
COMPOSITE BITE TURBO GEL

RELIANCE

TKO™
COMPOSITE BITE TURBO GEL

RELIANCE

TKO™
COMPOSITE BITE TURBO GEL

RELIANCE

TKO™
COMPOSITE BITE TURBO GEL

RELIANCE

TKO™
COMPOSITE BITE TURBO GEL

INSTRUCTIONS FOR USE

RELIANCE

TKO™
COMPOSITE BITE TURBO GEL

RELIANCE

TKO™
COMPOSITE BITE TURBO GEL

RELIANCE

TKO™
COMPOSITE BITE TURBO GEL

RELIANCE

TKO™
COMPOSITE BITE TURBO GEL

RELIANCE

TKO™
COMPOSITE BITE TURBO GEL

RELIANCE

TKO™
COMPOSITE BITE TURBO GEL

EN - ENGLISH

INDICATIONS FOR USE: TKO is a flowable light cure composite gel for bite turbos. Available in both pink and clear shades, the clear version offers a fluorescing monomer to aid in removal.

Performance Indicator: Bonds to enamel for duration of treatment.

INSTRUCTIONS FOR USE:

PROPHYLAXIS: Prophy the enamel surface. Rinse and dry thoroughly.

ETCH: Etch the enamel surface, rinse and dry.

APPLICATION OF GEL: Unscrew and remove the black storage cap from the tip pf the TKO syringe. Snap the needle tip dispenser in place and twist clockwise until needle is locked.

Dispense the desired amount of gel onto the etched enamel surface and shape as desired. Light cure for 20 seconds from close range with a dental curing light of 1000mw/2 or more.

⚠️ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

- ⚠️ Corrosive

⚠️ Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices. Protective eyewear should be worn by the patient and dental staff.

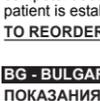
⚠️ **TEMPERATURE LIMITATION:** Store at room temperature (15-25°C).

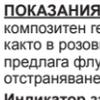
PATIENT POPULATION: The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

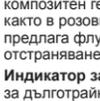
CONTRAINDICATIONS: Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

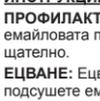
SAFE DISPOSAL: Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous- waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

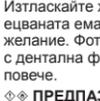
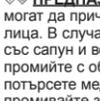
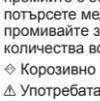
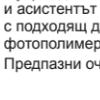
WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

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